

Harmony[®] Needle User's Guide

www.DialMedSupply.com



Integrated cap for aseptically removing the scab.



Clean button-hole site according to policy.



Tear pouch, remove Harmony[®] needle from package and inspect for damage or particles.



Without removing the cap, grasp the wings and cap together with the thumb and the forefinger.



Using the tip of the cap, remove button-hole scab.

Repeat disinfection procedure according to policy before attempting cannulation.



Remove cap.



Align Harmony[®] needle cannula with bevel facing up over the button-hole site.



Cautiously insert Harmony[®] needle into the established button-hole track ensuring the same angle and depth of insertion are used with each cannulation.



Observe pulsation of blood in the plastic tubing when the Harmony[®] needle is in the button-hole access.



Securely tape according to clinic policy and start treatment per facility procedures.

