

# WingEater

## Instructions For Use

### Step 1

Slide WingEater up until it just touches the wings



### Step 2

Place your hand flat on the site and WingEater with your last 3 fingers on the puncture site and your index finger behind the notch on the top of the WingEater



### Step 3

Gently pull the tubing until the needle is completely retracted into the WingEater, making sure the wings are below the safety line marked on the upper member of the WingEater



## Patient Assisted Hemodialysis Procedure

Grasp the WingEater with your thumb and index finger, placing your thumb in the indentation marked with the JMS logo. Pull the tubing until the wings have been fully retracted into the WingEater



JMS North America Corporation



Dial Medical Supply  
610-827-0987

fax 610-827-0987

[www.DialMedSupply.com](http://www.DialMedSupply.com)